SEL Tıbbi Malzemeler

ADRES : Fevzi Çakmak Mahallesi 10664.Sokak
No:38 / F Karatay / Konya / Türkiye
TEL : +90 332 324 30 80
FAKS : +90 332 324 30 81
WEB : www.vacusel.com
E POSTA : bilgi@vacusel.com

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VacuSEL® Sel Medical Supplies

SEL Medical Materials, founded as a company engaged in the trade in 2008, focused its activities on the medical materials in order to fill the gaps in the production of medical materials in Turkey by benefiting from the experience it has acquired.

Our company, which decided to start the production of vacuum blood collection tubes in 2013, contacted the world's leading manufacturers and received consultancy services from professionals who are experts in their fields. Considering the latest production Technologies, continuity of production, international quality standards, CE directives and market expectations, a special production model has been created to meet all of these expectations. Our production area has been specially built by specialist and experienced manufacturers of machinery and equipment to be used in production in parallel with the model created. Thus, the company has become capable of producing in a quality that can meet the increasing demands around the world by going beyond being able to manufacture medical equipment in Turkey under international standards only for Turkey.

Our company completed trial productions at the end of 2014 with its expert engineers, technicians and employees. By partnering with many companies that provide clinical laboratory services in both public and private sectors, our products have been used with great satisfaction in making diagnosis in such parts as Biochemistry, Hormone, Hematology, Immunology, Serology, Coagulation and they continue to be used with increasing demand. Our products work seamlessly with many different brands and models of analyzer devices. We are also in an effort to carry our quality further with our field visits and R&D studies. Interest in our products continues to increase day by day, both inside and outside the country. Our company has taken a worthy place in the market.

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- Potassium salt of Edta was used as Anticoagulant.
- The inner surface of the tubes was prepared by spraying K2 Edta and drying.
- It is produced in various volumes.
- It is generally used in hematology tests in clinical chemistry laboratories.
- K2 Edta, which is used as an active ingredient, prevents the coagulation process of the blood by blocking calcium ions in the blood.
- With EDTA tubes; Hemogram, Reticulocyte, Formula Leukocyte, Blood Group, Cross-match, Direct Coombs, Indirect Coombs tests are performed.
- In EDTA blood, blood smears can be made for erythrocyte and leukocyte counts, determination of hematocrit and the formula of leukocyte. However, EDTA blood is not suitable for the determination of electrolyte and calcium.

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- Potassium salt of Edta was used as Anticoagulant.
- The inner surface of the tubes was prepared by spraying K3 Edta and drying.
- It is produced in various volumes.
- It is generally used in hematology tests in clinical chemistry laboratories.
- K3 Edta, which is used as an active ingredient, prevents the coagulation process of the blood by blocking calcium ions in the blood.
- With EDTA tubes; Hemogram, Reticulocyte, Formula Leukocyte, Blood Group, Cross-match, Direct Coombs, Indirect Coombs tests are performed.
- In EDTA blood, blood smears can be made for erythrocyte and leukocyte counts, determination of hematocrit and the formula of leukocyte. However, EDTA blood is not suitable for the determination of electrolyte and calcium.

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- Tubes are prepared with 3.2% tri sodium citrate solution.
- It is produced in various volumes.
- It is generally used in clinical chemistry laboratories.
- 9NC; It is prepared to create 9 units of blood mixture for 1 unit of blood additive.
- PT, aPTT, Fibrinogen, D-Dimer, Coagulation Factors(FII, FV, FVII, FVIII, FIX, FX, FXI, FXII, FXIII), Collagen-ADP, Collagen-Epinephrine, Protein C, Protein S, ATIII, TT, Lupus Anticoagulant, VWF: Ag and VWF:RiCoF, Antifactor Xa are studied.
- Acid-base balance cannot be examined with citrated blood; it is not suitable for electrolyte and some calcium measurement methods. Citrate has a detrimental effect on the determination of some enzymes.

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- Tubes are prepared with 3.8% tri sodium citrate solution.
- It is produced in various volumes.
- It is generally used in clinical chemistry laboratories.
- 9NC; It is prepared to create 9 units of blood mixture for 1 unit of blood additive.
- PT, aPTT, Fibrinogen, D-Dimer, Coagulation Factors(FII, FV, FVII, FVIII, FIX, FX, FXI, FXII, FXIII), Collagen-ADP, Collagen-Epinephrine, Protein C, Protein S, ATIII, TT, Lupus Anticoagulant, VWF: Ag and VWF:RiCoF, Antifactor Xa are studied.
- Acid-base balance cannot be examined with citrated blood; it is not suitable for electrolyte and some calcium measurement methods. Citrate has a detrimental effect on the determination of some enzymes.

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- It is prepared without using any additives.
- It is produced in various volumes.
- Used in clinical chemistry laboratories.
- Serum tubes; It is used for many tests such as Biochemistry - Hormone - Microbiology - Serology - Immunology - Blood Bank.

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- The inner surface of the tubes was prepared by spraying clot activator.
- It is produced in various volumes.
- The clot activator used as an ingredient accelerates the coagulation process of the blood.
- It is generally used in clinical chemistry laboratories.
- Serum tubes; are used for many tests such as Biochemistry - Hormone - Microbiology - Serology - Immunology - Blood Bank.

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- In our tubes, a special gel that is resistant to high temperature and very stable in physical and chemical properties is used.
- Our gel effectively separates the serum from fibrin and cells. It also prevents substance exchange between blood cells and serum.
- It can keep the biochemical characters and chemical components of serum unchanged for a long time.
- The inner surface of the tubes was prepared by spraying clot activator.
- It is produced in various volumes.
- The clot activator used as an ingredient accelerates the coagulation process of the blood.
- Serum can be aspirated directly from the collection tube.
- It is generally used in clinical chemistry laboratories.
- Serum tubes; are used for many tests such as Biochemistry-Hormone-Microbiology-Serology-Immunology-Blood Bank.

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- The inner surface of the tubes was prepared by spraying Potassium Oxalate/Sodium Fluoride and drying.
- It is produced in various volumes.
- It is generally used in clinical chemistry laboratories.
- Potassium oxalate, which is used as an active substance, prevents the blood clotting process by blocking calcium ions in the blood.
- Sodium fluoride is used to prevent glycolysis in the blood.
- It is also known as Glucose Tubes. It is used in the long-term storage of some substances such as glucose measurement and ethanol. Antiglycolytic agents retain glucose for up to 5 days.
- The determination of hemoglobin in oxalate blood, erythrocyte and leukocyte counts, the determination of hematocrit can be done; plasma can be separated and used for many biochemical analysis.
- Acid-base balance cannot be examined with oxalate blood; it is not suitable for electrolyte and some calcium measurement methods. It has a detrimental effect on the determination of some enzymes such as oxalate, lactate and dehydrogenase. Oxalate blood cannot be used for smears, as it causes degeneration of the nuclei of leukocytes and notching of the edges of erythrocytes.

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- The inner surface of the tubes was prepared by spraying Lithium Heparin and drying.
- It is produced in various volumes.
- It is generally used in clinical chemistry laboratories.
- Heparin prevents blood clotting by specifically binding clotting factors IX and XI and by interacting with antithrombin III and increasing its ability to inactivate thrombin.
- Heparin tubes; are used for other biochemistry analysis requiring NT-ProBNP and Plasma. Sodium heparin is used for the lithium level. Sodium or lithium heparin is used for the ammonia level.
- Heparin blood can be used for the determination of electrolyte and erythrocyte fragility.

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- The inner surface of the tubes was prepared by spraying Sodium Heparin and drying.
- It is produced in various volumes.
- It is generally used in clinical chemistry laboratories.
- Heparin prevents blood clotting by specifically binding clotting factors IX and XI and by interacting with antithrombin III and increasing its ability to inactivate thrombin.
- Heparin tubes; are used for other biochemistry analysis requiring NT-ProBNP and Plasma. Sodium heparin is used for the lithium level. Sodium or lithium heparin is used for the ammonia level.
- Heparin blood can be used for the determination of electrolyte and erythrocyte fragility.

ÖZELLİKLER • PROPERTIES

Refemas Kodu Reference Code	Hacim Volume	Ebat Size	Açıklama Description	Paket / Koli (Adet) Package / Parcel (Pcs)
VSNH20	2 ml	13 x 75 mm	Sodyum Heparin, Pet, Steril <i>Sodium Heparin, PET, Sterile</i>	100 / 1200
VSNH30	3 ml	13 x 75 mm	Sodyum Heparin, Pet, Steril <i>Sodium Heparin, PET, Sterile</i>	100 / 1200
VSNH40	4 ml	13 x 75 mm	Sodyum Heparin, Pet, Steril <i>Sodium Heparin, PET, Sterile</i>	100 / 1200
VSNH50	5 ml	13 x 100 mm	Sodyum Heparin, Pet, Steril <i>Sodium Heparin, PET, Sterile</i>	100 / 900
VSNH60	6 ml	13 x 100 mm	Sodyum Heparin, Pet, Steril <i>Sodium Heparin, PET, Sterile</i>	100 / 900

Vakumlu ESR Tüpler Vacuum ESR Tubes

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- *Thrombin Serum Tubes are also known as Rapid Serum Tubes. It provides rapid coagulation with thrombin-based clot activator.*
- *The samples taken from patients who do not receive anti-coagulant treatment can be centrifuged after 5 minutes.*
- *It can keep the biochemical characters and chemical components of serum unchanged for a long time.*

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- *Thrombin Serum Tubes are also known as Rapid Serum Tubes. It provides rapid coagulation with thrombin-based clot activator.*
- *The samples taken from patients who do not receive anti-coagulant treatment can be centrifuged after 5 minutes.*
- *By separating the serum and blood cells with the gel barrier, a better quality and more stable sample is obtained.*
- *It helps to make analysis easier and faster since there is no need to work by transferring the sample to a second tube.*
- *In our tubes, a special gel that is resistant to high temperature and very stable in physical and chemical properties is used.*
- *Our gel effectively separates the serum from fibrin and cells. It also prevents substance exchange between blood cells and serum.*
- *It can keep the biochemical characters and chemical components of serum unchanged for a long time.*

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- VacuSEL Blood Collection Needle Tips are produced using equipment with the latest technology system in a way to provide the highest level of patient comfort.
- The surface of VacuSEL Blood Collection Needles is covered with silicone to reduce friction and pain to the patient, and the needle tip is laser sharpened to minimize pain. Thus, the vein is entered gently with less pain.
- Our needles are of a structure that will not hemolyze the blood sample taken. The needles have protective caps on both ends of them entering the vein and tube.
- There is a latex-free gray rubber valve at the end of the needle that enters the tube, allowing multiple samples from the same patient.
- Rubber valve is in a structure that will not leak blood during tube changes when more than one sample is taken.
- There are five different varieties: 18 G Pink, 20 G Yellow, 21 G Green, 22 G Black and 23 G Blue.

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- *VacuSEL Accessory Products meet all requirements at the highest level by supporting the user together with other products you need in blood collection.*
- *VacuSEL Holders are fully compatible with VacuSEL Needle and Blood Collection Tubes.*
- *Our holders are made of hard plastic and all sample tubes can easily fit into our holders.*
- *The tip of our holders is screwed so that normal and safe needle tips can be screwed into the holder completely and tightly.*
- *Our Safety Holders protect healthcare workers against the risk of needlestick*

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Why VacuSEL?

Advantages of VacuSEL Vacuum Blood Collection Tubes:

- Hygienic, safe and comfortable blood sample collection thanks to the vacuum system
- Valid test results thanks to the correct blood-additive mixing ratio
- Consistent, optimally and correctly filled tubes at all times
- Vacuum adjusted for precise fill volumes
- Standard color-coded caps to aid identification
- Low contamination risk thanks to the closed system
- High quality and innovative products
- Resource (supplier) diversity
- Perpetual Stock
- Fast and flexible delivery
- Cost Optimization
- Reliable service
- Technical support for laboratories
- Staff education